

This sheet is skin developed for medical students and internships to simulate. Its texture is more similar to human skin than conventional products, allowing more realistic practice.

Be sure to read this manual before using this product to ensure safe use over a long period of time.

Items Included:

Check the contents of this package.

- (1) BRP-L (Multipurpose skin) -----1
- (2) Instruction manual (this document) -----1

Directions for Use:

- (1) The user can freely cut and use this product.
- (2) Electric knife cannot be used.
- (3) Keep this product away from fire.
- (4) You can color it with makeup tools.
- (5) You can use a make-up remover to remove the color of your make-up tools.

Precautions for Use and Storage Instructions:

- (1) Do not use an electric scalpel.
- (2) Do not stitch too tightly when suturing. This may cause the “BIODY” to tear.
- (3) Do not store this product in direct sunlight, high temperatures or high humidity for long periods of time. This may cause deformation or discoloration.
- (4) Do not boil this product or immerse it in hot water (over 60°C). Keep this product away from fire.
- (5) Do not use oil-based markers or felt-tip pens. The color will penetrate the material and cannot be removed.
- (6) Do not use chemicals such as organic solvents or acidic or alkaline chemicals. Never use solvents such as paint thinner or chloroform.
- (7) If the surface gets dirty, wipe it with an alcohol such as ethanol.
- (8) Be careful when using it with other resins. Resins may chemically react with each other.